

Instructions for Use of the Gel

Activ'M Gel® is a silicone gel with a matte effect, composed of a new generation of medical-grade silicones. These components were selected for their biocompatibility.

Activ'M Gel® is used in the treatment of hypertrophic and keloid scars, and in aesthetic enhancement: it visibly reduces your scars, whether they are recent or old.

Activ'M Gel® has the following beneficial effects:

- Prevention
- Healing

Activ'M Gel® is mainly used on the face and in bending zone areas and on body joints where silicone dressing is not easy to apply. After application Activ'M forms only a protective and invisible film on the skin.

USES

The intrinsic qualities of the silicone gel Activ'M Gel® improve appearance of old and new scars.

- They cure hypertrophic or keloid scars, whether recent or old
- They prevent closed wounds from becoming keloid or hypertrophic scars (following surgery or burn injuries).
- Activ'M Gel® can be used on children's scars.

CONTRAINDICATIONS

Activ'M Gel® cannot be used in the following cases:

- Open wound (following surgery or burn injuries),
- Maceration,
- Mucosa and close to eyes,
- Do not inhale or ingest Activ'M Gel®.

In event of skin reaction, suspend treatment and consult your doctor.

SECONDARY EFFECTS

No known side effects.

DIRECTIONS

- Before use, clean and dry the scar thoroughly (using a neutral soap).
- Apply a small quantity of Activ'M Gel® directly to the scarred area. Allow to dry for a few seconds.
- Activ'M Gel® must be applied twice daily – morning and afternoon.
- Put the top back on the tube carefully after use.

It is recommended to continue the treatment for 2 to 4 months minimum. Treatment can be prolonged for up to 1 year

With friction, a part of the silicone protective film can be removed. As long as a fine amount of the silicone film remains on the skin, leaving on your skin with a silky feel.